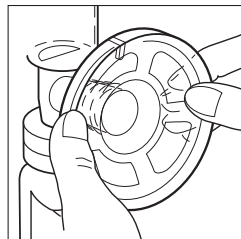
1

2

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MOM INSPIRED. HOSPITAL TRUSTED.

EN Flexishield Areola Stimulator Instructions

The Flexishield device is an option designed to provide increased stimulation and encourage the let-down reflex. It works by compressing the areola area which helps stimulate the milk-ejection reflex. Flexishield may be used with the Ameda HygieniKit or the Ameda One-Hand Breast Pump (breast flange, 25 mm).

Before Each Use

Wash your hands with soap and water. 

Sanitize

Boil the Flexishield in water for 20 minutes before the first use and then once a day. After each use, wash the part in hot, soapy water, rinse well with clean water and fully air dry.

Using the Flexishield

1. Squeeze together the sides of the outer circular section of Flexishield by pushing with the thumb and fingers. Insert Flexishield into the breast flange opening, as shown. The ribs on the inserted part of Flexishield should contact the side walls of the breast flange opening.
2. Seal Flexishield to the breast flange opening by locking the edge of the Flexishield over the rim of the breast flange opening using the palm of your hand or thumb and forefinger.

Flexishield is now ready to use.

For easy removal, the tab on Flexishield stimulator's outer edge prevents a seal from forming with flange.

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Ameda, Flexishield, HygieniKit and associated logos are trademarks of Evenflo Company, Inc. ©2009 Evenflo Company, Inc.